

## **Specifications**

#### A1cChek Pro Glycohemoglobin Analyzer


, rochon roc clyconomograph, maly 201				
Test item	Glycohemoglobin (HbA1c)			
Principle	Boronate affinity chromatography			
Measuring range	4.0% ~ 14.0%			
Sample material	Venous or capillary blood			
Sample volume	3μL (whole blood)			
Testing time	5 min			
Throughput	5 min to the first result, subsequently 1 result every 1 min			
Result metric	NGSP (%); IFCC (mmol/mol); eAG (mg/ml; mmol/L)			
Voice prompt	Voice prompt in the whole process of operation			
Printer	Internal thermal printer			
Data storage	1000 results			
Data port	USB, RJ45 LAN, RS232-C			
Wireless connectivity	WIFI			
Power supply	External power adapter			
AC input	100-240V ~ , 1.5A, 50-60Hz			
DC output	19.0V , 4.74A			
Temperature calibration	Automatic calibration by temperature sensor			
Dimension(mm)	(341±2)mm × (266±2)mm × (234±2)mm			
Screen size	154.1mm × 85.9mm			
Weight of analyzer	4.4Kg			
Working condition	Temperature: 10 ~ 40°C, Humidity: 30% ~ 75%, Atmospheric pressure: 700hpa ~ 1060hpa			
Storage condition	Temperature: -20 ~ 55°C, Humidity: <80%, Atmospheric pressure: 700hpa ~ 1060hpa			

### Catalog

Product	Catalog No.	Contents	
A1cChek Pro Glycohemoglobin Analyzer	A1C-M31	A1cCheck Pro Glycohemoglobin Analyzer Power Adapter User Manual Quick Guide Cleaning and Maintenance Guide Warranty Hangtag	
A1cChek Pro Glycohemoglobin Test Kit	A1C-S32	HbA1c Test Strip Sampler Code Chip Package Insert Buffer A Buffer B Blood Collector	200 200 2 1 35ml x 1 35ml x 1 200

Wuxi Biohermes Bio & Medical Technology Co., Ltd.

11th Floor,530 Mansion, #18 Qingyuan Road, Xinwu District, Wuxi 214135, China
Tel: +86-510-85385897 Email: info@biohermes.com.cn; servi@biohermes.com.cn

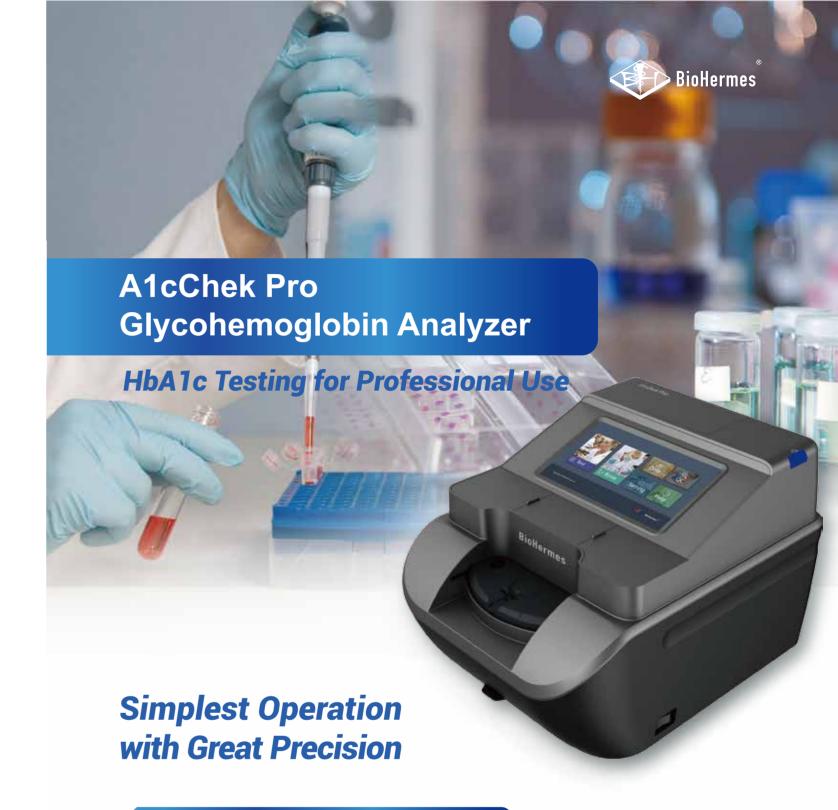

Accurate Result with CV<3%

**One-Step Operation** 

4 Samples / 8 Minutes










# **Clinical Application**

### What is HbA1a

- 1. HbA1c, formed in a non-enzymatic glycation pathway by hemoglobin's exposure to plasma glucose, reflects average glycemia over several months.
- 2. As a primary technique to assess the effectiveness of diabetes management, HbA1c has strong predictive value for diabetes complications. Lowering HbA1c has been shown to reduce complications.
- 3. HbA1c  $\geq$ 6.5% (48 mmol/mol) is one of the criteria for diabetes diagnosis. Normal HbA1c range is 4.0-5.7% (20-39 mmol/mol), while 5.7-6.4% (39-46 mmol/mol) is considered as prediabetes.
- 4. Point-of-care testing for HbA1c provides the opportunity for more timely treatment changes.

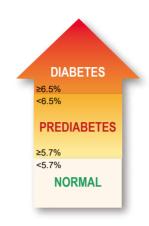

### One-step Operation

#### **Venous Blood**





Turn the anti-coagulated venous blood tube upside down for at least 6 times. Use the blood collector to dip the blood sample from the tube , and touch the sampler thread to the blood sample until the thread becomes completely red.

#### **Capillary Blood**





Put the lancing device against the sampling site, press the release button on lancing device to prick the fingertip. Touch the sampler thread to the blood sample until the thread becomes completely red.

#### **Apply the Blood Sample**





Press and hold the sampler on the sampler port for 3 seconds. After three beeps, remove the sampler.


## **Accurate Results**


### Comparison Evaluation Report



### Shanghai Sixth People's Hospital


CV (%)	Sample 1 (5.2% HbA1c)	Sample 2 (11.6% HbA1c)
Within-run	1.9%	1.8%
Between-run	0.0%	0.6%
Between-day	1.4%	0.0%
Total	2.3%	1.9%

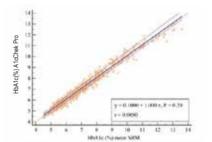


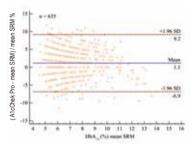

## **BCCL** Beijing Center for Clinical Laboratory

Mean Value (HbA1c %)	SD	CV
5.58	0.15	2.60%
11.88	0.24	2.04%

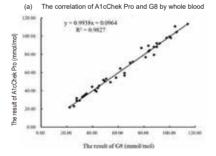

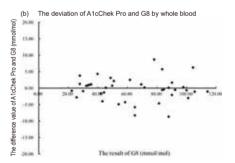

### 17 Patents of 12 Countries

