



GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD.

Wheelchair DY1903



Size(cm) Model													
	(cm)	(cm)	(cm)	(cm)	(cm)	(cm)	(cm)	(cm)	(cm)	(kg)	(cm)	NW GW	
DY1954LGC	64	28	46	60	20	50	88	102	41	41	100	80*27*89 1PC/CTN	18/20





USER'S MANUAL



- | | | | |
|-----------------------|----------------|----------------|----------------|
| 1. Hand Grip | 2. Backrest | 3. Armrest pad | |
| 4. Detachable armrest | 5. Hand rim | 6. Brake Lever | |
| 7. Rear Wheel | 8. Legrest | 9. Fork | |
| 10. Front Castor | 11. Side Panel | 12. Cross Tube | 13. Foot Plate |



How to Assemble and Detach the Wheelchair

1). Armrest

A. Press the armrest button.



B. Lift Up the Armrest with both hands.



2). Legrest (From Left to Right)


A. Lift up the foot plate;

B: Pull the Swing Pin to the opposite side;

C. Turn the legrest as the blue arrow indicates;

D. Lift Up the Legrest.

E. Done.

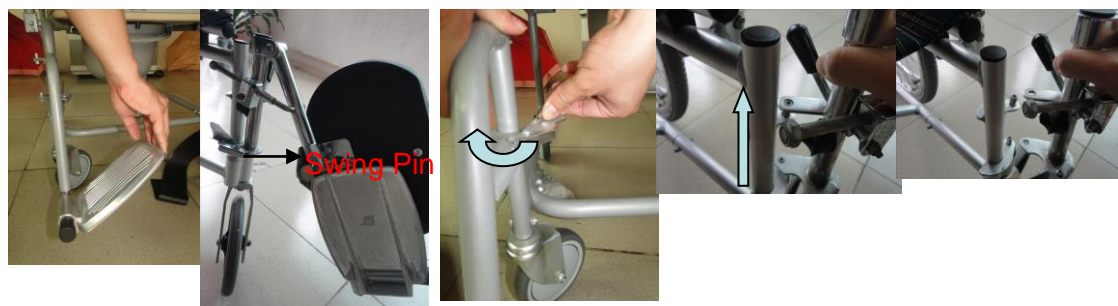



How to Lock and Unlock the Wheelchair

It is very important to lock the Wheelchair when simply sitting on it.

- A. This is how the brake looks like when the wheelchair is unlocked.
- B. Hold the brake lever and move it forwarder, as the arrow indicates.
- C. When you hear a “Snap” , the wheelchair is locked.
- D. This is how the brake looks like when the wheelchair is locked.

To unlock the wheelchair is operate the other way around.



How to fold the Wheelchair

- A. Take out the head rest and footrest before you fold the wheelchair.
- B. Pull up the string on both sides of the seat frame, and the wheelchair will fold as you pull.

